World of Microsurgery  
*Welt der Mikrochirurgie*

Innovation & Quality

S&T®

**S&T<sup>®</sup>**  
Confidence  
*Vertrauen*

All  
**S&T<sup>®</sup>** instruments  
carry a lifetime  
warranty against  
defects in  
workmanship  
and materials.

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**S&T<sup>®</sup>**

All instruments are subject to technical changes in design and construction. We will supply the current model.

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Our general terms and conditions of sale and deliveries are applicable.

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Reprints of this catalogue in whole or in part is not permitted. Printed or electronic matters of any kind for which illustrations of our products are required with or without the respective article number are subject to our written consent.

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Certification Mark:

Scope of Certificate:

Production of microsurgical sutures  
and non active microsurgical instruments

Applied Standard(s):

EN ISO 13485:2003  
Medical Devices –  
Quality Management Systems –  
Requirements for regulatory purposes

The Certification Body of TÜV PRODUCT SERVICE GMBH certifies that the company mentioned above has established and is maintaining a quality system which meets the requirements of the listed standards. See also notes overleaf.

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## To our Customers *An unsere Kunden*

### Dear Customer...

welcome to **S&T**'s latest catalog — your resource and authority in the field of Microsurgery.

Microsurgery has evolved through the years, with new procedures pioneered and new techniques perfected. **S&T** has been at the forefront, accommodating each new innovation with skillfully engineered instruments. Quite simply, we've been on the cutting edge of the cutting edge.

The growing field of Microsurgery depends on more than just the right instruments. It depends on research, knowledge and expert advice.

**S&T** is proud to acknowledge its collaboration with the world's most prestigious microsurgeons. Together, we have crafted some of the most innovative products in the field. **S&T** products are crafted from premium materials, and are built from a sound foundation of research, expert cooperation, and precise engineering. We at **S&T** are so confident in our products and processes that we challenge you to find a better line of instruments anywhere else in the world.

**All **S&T** instruments carry a lifetime warranty against defects in workmanship and materials.**

We thank you for your loyalty and support. Our pledge to you is that **S&T** will continue its dedication to excellence — so you can achieve it.

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## Selecting your Instrument

### Wahl der Instrumente

## Selecting the Appropriate Instrument

All **S&T** products must be used exclusively for their intended purpose.

In this catalog you will find information concerning the appropriate use of each instrument. The following highlights are some of the important factors to keep in mind when selecting the instrument that best suits your needs:

## Instrument Length

When deciding which length of instrument to use, careful consideration should be given to the depth of the surgical field within which you will be working. For example, instruments 11 centimeters in length can only be used comfortably on or very near the surface. Each additional centimeter of the instrument enables you to work that much deeper. Our longest instruments are designed for procedures that involve deep surgical fields, or in areas where it's not practical to use shorter instruments. Personal preference may also be a consideration as surgeons with larger hands may feel more comfortable using longer instruments. In general, instruments that are 15 cm or shorter are commonly used for hand surgery, while instruments 18 cm or longer are typically used in procedures involving free tissue transfer (free flaps).

## Round or Flat Handle?

We generally recommend the use of round handle needleholders and scissors. Round handle instruments facilitate the rotating movement needed when using a needleholder for passing sutures through tissue. It has been our observation that most surgeons prefer the feel of round handle instruments.

Flat handle, Jewelers-type forceps are the preferred style for both forceps and dilators. A full range of flat handle instruments are found within this catalog.

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**S&T<sup>®</sup>**  
Selecting your  
Instrument  
*Wahl  
der Instrumente*

## **Advantages of Balanced Instruments**

Balanced instruments are designed to put the center of gravity (the balance point) in the web space between the thumb and index finger. This is accomplished by adding a slight counter weight to the proximal end of the instrument. The design of balanced instruments enhances tactile feel, aiding in better control and precision. Balanced instruments are less tiring to handle, especially during lengthy procedures. Round handle balanced instruments have better handling characteristics than the standard (non-balanced) round handle instruments.

## **Angulated and Curved Forceps**

Many instruments feature angulated or curved tips, giving you a grip that is parallel/perpendicular to the working surface, depending on the object you are handling. While most surgeons are comfortable with our standard 45° or 36° angles, others may prefer the flexibility and curvature of an instrument that conforms to any hand position. Curved forceps are designed to facilitate needle driving or passage through tissue. Straight forceps are commonly held in the non-dominant hand and are used to either grasp tissue or receive the needle when suturing.

## **Instrument Sets**

In response to our customers' needs, we have developed sets of instruments made up of the most commonly used microsurgical instruments. These sets are available in both round and flat handle versions. See catalog chapter 2.

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# Care of Instruments *Instandhaltung der Instrumente*

## Instrument Care and Handling

Proper care and handling techniques will help preserve the life of an instrument, and ensure that it functions at peak performance. Avoid having your instruments lay loose together. You should not combine delicate micro instruments with macro instruments as this may cause damage to the delicate tips. Most damage to microsurgical instruments occurs during processing and cleaning, so it is crucial that you handle the instruments with great care. Finally, we recommend you invest in a sturdy instrument container that is specifically designed for the storage, sterilization and transport of microsurgical instruments.

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## Decontamination

### Warning!

Most instrument damage occurs during processing and cleaning. The best way to avoid this is by handling them with great care. This is especially true for microsurgical instruments.

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## Pre-cleaning

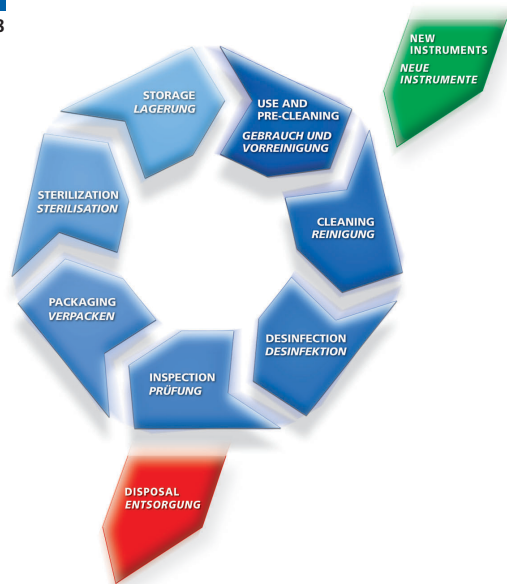
It is best to remove excess tissue or other residues from your instruments as soon as possible, ideally immediately after the procedure. This can be done manually or by an ultrasonic cleaner. We suggest that a disinfecting agent be used. Keep in mind that temperatures below 45° C are needed to avoid protein coagulation.

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Make sure to move all approximators, ratchets and sliding parts to ensure that all residues are removed. Clamps, scissors and needle holders need to be opened to allow contact with cleaning solutions.

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## Cleaning

In general, you should follow the prescribed guidelines established for your institution. It is recommended that cleaning and disinfecting be done in a thermal disinfectant.

Initial cleaning commences at temperatures of approximately 40-60° C with pH-neutral cleaners (preferably with enzymatic agents). Rinsing is best done with cold water and without additives.

Foam generation should be avoided as it may inhibit the cleaning action.

You should follow the manufacturers' instructions regarding use of cleaning agents and cleaning equipment. Water used in all cleaning processes should have a low concentration of chloride ions, as excessive concentrations may lead to pitting and other forms of corrosion.

Final rinsing should be done with distilled / deionized water. This helps to avoid staining of the instruments.

**To avoid damaging the delicate tips of micro instruments, process them in their own designated tray. Avoid overloading the tray, making sure that micro instruments do not come in contact with macro instruments. Scissors and needle holders should be opened to ensure proper cleaning.**

Be mindful of instruments with large surface areas. They can overshadow smaller instruments situated below them, inhibiting them from being cleaned properly and effectively.

Instruments should be dried thoroughly after cleaning, to avoid corrosion.

Only clean instruments should be sterilized.

**Note: New instruments should be processed and sterilized prior to their first use.**

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## Disinfection

For the disinfecting process, we recommend using a cycle with the highest temperatures (approximately 93°C).

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## Inspection

Instruments should be inspected after each use and immediately after cleaning. The instruments tips should approximate well. Ensure that all scissors and needleholders are in proper working order. Pay special attention to hinges and box locks. Make sure that they are thoroughly cleaned and function smoothly. Check for bent and broken tips. Instruments that are damaged should be removed from service and either be repaired or replaced.

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Instruments that are beyond repair should be discarded, ensuring that they will not find their way back into the operating room.

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If corrosion is apparent, the instrument should be removed from service and sent out for repair. Depending on the nature of the corrosion, some mild forms may be polished away. In the case of serious pitting or corrosion, the instrument should be discarded as to avoid cross contamination with other instruments.

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## Lubrication

Instruments with moving parts perform best when properly lubricated. Instrument lubrication not only makes an instrument perform better, but also reduces the effects of corrosion, ultimately prolonging the life of the instrument. We recommend any paraffin-based lubricant that is certified for use with medical devices. Remember to use only enough to wet the relevant areas, allowing instruments to drip dry before placing them in a tray or wrapping.

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# S&T<sup>®</sup>

## Care of Instruments *Instandhaltung der Instrumente*

S&T instruments have a **lifetime warranty** on workmanship and materials. Instruments that are judged to be defective should be returned to **S&T**. Contact your local representative. When returning an instrument, supply as much of the following information as possible: Article code or catalog number; numerical code, located next to the CE mark; and description of the defect. This information is used to improve **S&T** instruments and ensure that our products remain state-of-the-art.

### Packaging

Packaging plays an important role in the sterility of your instruments. Corrosion may occur if instruments are handled improperly during the packaging process, as the drying cycle in the sterilizer may be inhibited. Further, it is possible to invalidate the entire sterilization process if defective packaging materials do not properly protect the sterilized instruments.

### Sterilization

In general, you should follow your institution's set guidelines and procedures when sterilizing microsurgical instruments. The most common form of sterilization is steam autoclaving. Microsurgical instruments should be sterilized in their own case. We recommend autoclaving your **S&T** instruments in saturated steam, at a minimum of 134°C, for 12-18 minutes. Stained instruments are an indication of problems having to do with cleaning/sterilization processes relating to one or more of the following:

- cleaning agents
- steam quality in sterilization
- water quality
- other agents

**Warning:**  
**Non-stainless or corroded instruments may damage other instruments during sterilization.**

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# Taking Care of Instruments *Instandhaltung der Instrumente*

## Storage

Storage sites should be clean, dust-free and well ventilated. Environmental parameters such as temperature, humidity and radiation should be congruent with the standards for human habitation. Humidity must be below the dew point, at all times.

## Use

The life of an instrument may be extended if it is cleaned as often as possible during a procedure. Blood and other contaminants should be removed from the surface of the instrument before it is allowed to dry. This is best done with distilled /deionized water, **not** with saline or ringers solution.

Instruments that appear or become damaged during a procedure should be removed from use and marked for later evaluation, repair and/or replacement.

Certain instruments supplied by **S&T** have special cleaning instructions. In such cases, refer to the cleaning manual included with the product or contact us for a replacement copy of the instructions.

An influencing factor in an instrument's functionality is the transportation of the instrument between processing facilities, storage and use. We recommend the user evaluate methods and establish a system that is as simple and safe as possible.

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**Important:**

The guidelines that we have provided here are general in nature. Care and handling procedures vary from country to country and from institution to institution. You should adhere to procedures established by your local protocol. If you require additional information, contact us directly or contact your local distributor.

**Service partners**

**S&T** has service partners all over the world. You may contact our partners for answers to questions relating to our products and services (such as workshop and training course support for microsurgical technique, or teaching/training materials for the care and handling of surgical instruments). For a list of our business partners and products as well as services we refer you to our web site

**[www.microsurgery.ch](http://www.microsurgery.ch)**

Of course you may also contact us directly.

**Phone: +41 52 672 74 07**

**S&T AG**  
**Zollstrasse 91**  
**CH-8212 Neuhausen**  
**Switzerland**

It will be our pleasure to see to all your needs in instrumentation. Modifications and ideas for new instruments are always welcome.

**Please note:**

The matrix with application information supplied in the product sections are to be taken as advice. This information has been supplied by surgeons who use the instruments as specified. The use of the instruments is not limited to these applications, and that other uses are not only possible but sensible. We would appreciate it if you could inform us of such additional fields of application you may have found.

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fax  
**+ 41 52 672 7402**

*Telefax*  
***+41 52 672 7402***

Satisfied customers are loyal customers.

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Thank you.

*Herzlichen Dank.*

# Contents Inhalt

	<b>Sutures for Microsurgery</b>	<b>1.01-06</b>	<b><i>Nahtmaterial für die Mikrochirurgie</i></b>	<b>1.01-06</b>
	<b>Instrument Sets</b>	<b>2.01-16</b>	<b><i>Instrumentensätze</i></b>	<b>2.01-16</b>
	Basic Lab Set	2.01-02	<i>Labor-Grund-Set</i>	2.01-02
	Basic Hand Surgery Set	2.03-06	<i>Handchirurgie-Grund-Set</i>	2.03-06
	Basic Reconstructive Surgery Set	2.07-10	<i>Wiederherstellungschirurgie-Grund-Set</i>	2.07-10
	Perforator Flap Set by Blondeel	2.11-12	<i>Perforator Lappen-Set nach Blondeel</i>	2.11-12
	Combined Basic Hand- and Reconstructive Surgery-Set	2.13-16	<i>Kombiniertes Hand- und Wiederherstellungschirurgie-Grund-Set</i>	2.13-16
	<b>Scissors</b>	<b>3.01-10</b>	<b><i>Scheren</i></b>	<b>3.01-10</b>
	<b>Forceps</b>	<b>4.01-16</b>	<b><i>Pinzetten</i></b>	<b>4.01-16</b>
	Classic Line forceps	4.01-08	<i>Classic Line Pinzetten</i>	4.01-08
	Balance forceps BF®	4.09-12	<i>Balance Pinzetten BF®</i>	4.09-12
	Forceps by Banis	4.13-14	<i>Pinzetten nach Banis</i>	4.13-14
	Tubing Introducer forceps / Tissue Forceps	4.15-16	<i>Führungspinzetten / Gewebepinzetten</i>	4.15-16
	<b>Vessel dilators</b>	<b>5.01-06</b>	<b><i>Gefäßdilatoren</i></b>	<b>5.01-06</b>
	Vessel dilator forceps	5.01-04	<i>Gefäßdilator Pinzetten</i>	5.01-04
	Special vessel dilators	5.05-06	<i>Spezial-Gefäßdilatoren</i>	5.05-06
	<b>Needleholders</b>	<b>6.01-08</b>	<b><i>Nadelhalter</i></b>	<b>6.01-08</b>
	Standard needleholders	6.01-04	<i>Standard Nadelhalter</i>	6.01-04
	Needleholder forceps	6.05-06	<i>Nadelhalterpinzetten</i>	6.05-06
	Double action needleholders	6.07-08	<i>Doppelgelenk Nadelhalter</i>	6.07-08
	<b>Micro Clamps</b>	<b>7.01-10</b>	<b><i>Gefäßklemmen</i></b>	<b>7.01-10</b>
	Acland clamps	7.01-10	<i>Acland Klemmen</i>	7.01-10
	Single use micro clamps	7.07-08	<i>Einweg-Mikroklemmen</i>	7.07-08
	Clamp applying forceps	7.09-10	<i>Klemmenanlegepinzetten</i>	7.09-10
	Pener nerve approximator	7.09-10	<i>Pener Nerven Approximator</i>	7.09-10
	<b>Instrument cases</b>	<b>8.01-10</b>	<b><i>Instrumentendosen</i></b>	<b>8.01-10</b>
	Instrument cases	8.01-06	<i>Instrumentendosen</i>	8.01-06
	Instrument Racks	8.07-08	<i>Instrumentenhalter</i>	8.07-08
	Clamp Box / Instrument Beaker	8.09-10	<i>Klemmendose / Bereitschaftsdose</i>	8.09-10
	<b>Special Instruments and Accessories</b>	<b>9.01-16</b>	<b><i>Spezialinstrumente und Zubehör</i></b>	<b>9.01-16</b>
	Arteriotomy clamp	9.01-04	<i>Arteriotomieklemme</i>	9.01-04
	Hand surgery	9.05-08	<i>Handchirurgie</i>	9.05-08
	Peripheral nerves and tendons	9.09-14	<i>Periphere Nerven und Sehnen</i>	9.09-14
	Instrument maintenance	9.15-16	<i>Instandhaltung</i>	9.15-16
	<b>Index</b>	<b>10.01-08</b>	<b><i>Index</i></b>	<b>10.01-08</b>
	All Products from A to Z	10.01-08	<i>Alle Produkte von A bis Z</i>	10.01-08

0.15